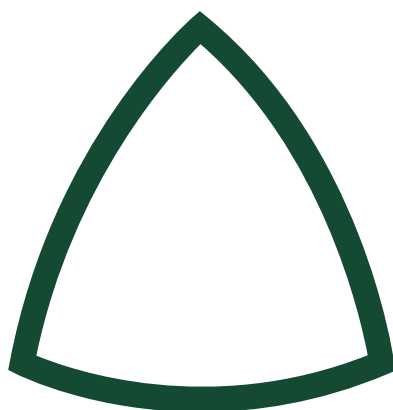
Prostasella[®]

BG	1	ES	19	IT	37	PT	55
CS	4	ET	22	LT	40	RO	58
DA	7	FI	25	LV	43	SK	61
DE	10	FR	28	NL	46	SL	64
EL	13	HR	31	NO	49	SV	67
EN	16	HU	34	PL	52		

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E-mail: service@4hnp.com

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Instructions for Use

Class I medical device in accordance with Regulation (EU) 2017/745 on medical devices (MDR).

Prostasella

Ergonomically shaped bicycle saddle with a pressure-relieving design to reduce pressure in the perineal and prostate areas.

Product	REF	Item number	UDI	Unique Product Identifier	PZN	number
			UDI-DI (G-TIN13)			
Prostasella S		4HPS001		04270004735505		19985122
Prostasella M		4HPS002		04270004735512		19985139
Prostasella L		4HPS003		04270004735529		19985151
Prostasella XL		4HPS004		04270004735543		19985168
Prostasella RPx		4HPS005		04270004735536		19985240

Intended Use

The Prostasella bicycle saddle is an accessory for bicycles and e-bikes designed to provide ergonomic seating support while riding.

The special saddle shape is designed to reduce pressure in the perineum and prostate area.

Medical Benefits

The ergonomic design can help:

- reduce pressure in the perineal area
- improve comfort while cycling
- reduce strain on the prostate and pelvic area

Intended users

The saddle is intended for adult cyclists, recreational and everyday riders, and for use on bicycles and e-bikes.

The product is intended for personal use and is not intended for children.

Intended environment

The saddle is intended for:

- Road traffic
- Recreational riding
- Cycling training

Indoor or outdoor use.

Contraindications

The product must not be used:

- if the saddle or saddle frame is damaged
- if the mounting hardware is damaged
- if installed improperly
- if the permissible load limit is exceeded

Individuals with existing medical conditions in the pelvic or prostate area should consult a doctor before use.

Warnings and Precautions

WARNING – Risk of Fall

A damaged or improperly installed saddle can cause the structure to fail and result in serious falls.

Do not use if:

- There are cracks in the saddle
- The saddle frame is bent
- The mounting bolts are loose

WARNING – Incorrect Installation

- Install only on compatible seatposts
- Secure only within the specified clamping range

- Follow the manufacturer's torque specifications

WARNING – Overloading

Exceeding the load limit may cause the saddle to fail.

Installation Instructions

Use only seatposts with a 7 mm mount.

1. Loosen the old saddle clamp.
2. Insert the saddle frame into the saddle clamp within the marked area.
3. Center the saddle.
4. Tighten the bolts evenly.
5. Set the torque according to the seatpost manufacturer's specifications.
6. Check for stability.

Take a test ride after installation. If you are unsure, have the installation performed by a specialist dealer.

Usage

Before every ride:

- Check that the saddle is securely fastened
- Check for damage
- Check the bolts

Do not use the product if any abnormalities are found.

Adjustment

Comfort and pressure relief depend heavily on the correct adjustment.

Recommended adjustment:

- Adjust the saddle height correctly
- Set the angle to horizontal or slightly forward
- Adjust the position according to your height

If you are unsure about the settings, please have them adjusted by a specialist dealer.

Maintenance and Inspection

Regularly inspect the saddle surface, the saddle frame, and the mounting bolts. If damaged, the saddle must be replaced immediately.

Service life

Service life depends on frequency of use, load, and environmental factors. Replacement is required if visible wear or material fatigue is observed.

Cleaning

Clean with water and a mild detergent. Do not use solvents, harsh chemicals, or high-pressure washers.

Storage

The saddle should be stored in a dry, shaded area at room temperature (–10 °C to +40 °C) and protected from direct sunlight.

Transport

During transport, the saddle should be protected from heavy mechanical stress. Do not place heavy objects on the saddle.

Side effects

In rare cases, the following may occur:

- Pressure sores
- Pain in the buttocks
- Temporary discomfort in the pelvic area

These may be caused by incorrect adjustment or an unsuitable sitting position.

If unexpected side effects or complications occur during or after use that are suspected to be related to the Prostasella product, please contact 4H Entwicklungen GmbH immediately at info@4hhp.com or the relevant authority in your country.

Disposal

The saddle can be disposed of in accordance with local regulations for metal and plastic parts.

CE Marking

This product complies with the requirements of Regulation (EU) 2017/745 on medical devices (MDR).

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4H Entwicklungs GmbH